# NECCHI

## N5200

- GB System for aerotherapy portable, ultrasound method
- D Tragbares system für aerosoltherapie mit Ultraschall
- F Système d'aérosolthérapie portable, à ultrasons
- E Sistema para aerosolterapia portátil, de ultrasonidos
- P Nebulizador portátil por ultra-sons para aerosolterapia
- SLO System za aerosol terapijo prenosnik, na ultrazvok
- HR Prenosivi sistem za aerosol terapiju sa ultrazvok
- PL Aparat do aerozoloterapii przenosny, ultradźwiękowy
- H Aeroszol terápia rendszer hordozható, ultrahangos

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## GENERAL INTRODUCTION

The Necchi aerosol therapy device is one of the most effective instruments available for treating common ailments of the upper and lower respiratory tract (nose, throat, bronchial tubes and lungs). The device is capable of nebulising any type of drug quickly and effectively. Ultra-compact, practical and portable, it can be taken with you wherever you go.

The device can also be equipped with a set of accessories including a car power outlet adapter, meaning that it can even be used to perform aerosol therapy easily and conveniently when away from home.

The device uses ultrasound technology for extremely quick and quiet treatment, making it the ideal solution for treating people while they are calm and relaxed. The simple, linear design is specifically intended to make therapy easier and more practical, without sacrificing quality and safety. The device is practical, easy to handle, safe to use and requires no complex maintenance.

It is the ideal solution for anyone who needs to perform two or more aerosol therapy sessions per day or who travels frequently. The device must only be used with drugs prescribed by your doctor.

This device is designed and built using the latest technology. The device is equipped with a double valve system for more effective inhalation, reducing drug wastage to a minimum. The device also comes with a complete set of accessories.

### Characteristics:

Portable with dual home/car function  
Ultra-compact, light and easy to use  
Extremely quiet (noise levels less than 40dB)  
Adjustable nebulisation intensity  
High nebulising speed (up to 1.0 ml/minute)  
Reduced therapy times  
Drug container ampoule capacity: 3 ml  
Ultrasound frequency: 2.5 MHz

Sharing accessories with others is not recommended. Read the instructions thoroughly before use and keep for future reference.

## GENERAL PRECAUTIONS

When using the device, carefully follow the safety instructions given below to prevent the risk of damage to property or personal injury and to avoid seriously damaging the device itself. Keep the instructions for future reference.

## PRECAUTIONS

- This device must only be used for the purpose for which it is intended, specifically, for aerosol therapy.
- Any other use is inappropriate and, as a result, dangerous. The manufacturer cannot be held liable in the event of damage caused by improper, incorrect or unreasonable use, or if the device has been connected to an electrical system that is not in compliance with applicable standards.
- If the power cable appears damaged, to prevent the risk of danger it must only be replaced by the manufacturer, an authorised technical support centre or by a qualified electrician.
- Do not use the device in the presence of anaesthetic mixtures which become potentially flammable in contact with air, oxygen or nitrous oxide.
- Disconnect the device from the main socket before filling.
- Do not overfill beyond the correct level.
- Always disconnect the device from the mains socket after every use.
- Keep the device and its power cable away from sources of heat.
- Do not use the device while in the bath, in excessively damp conditions or near bathtubs, basins etc.
- Do not drop or immerse the device in water or any other liquid.
- Never use the device while sleeping or if you feel drowsy.
- This device must not be used by children or persons with diminished mental, physical or sensorial capacities, unless said persons have received specific assistance or instruction for using the device by a person responsible for their safety.
- Children must be supervised during treatment to prevent them from playing with the device.

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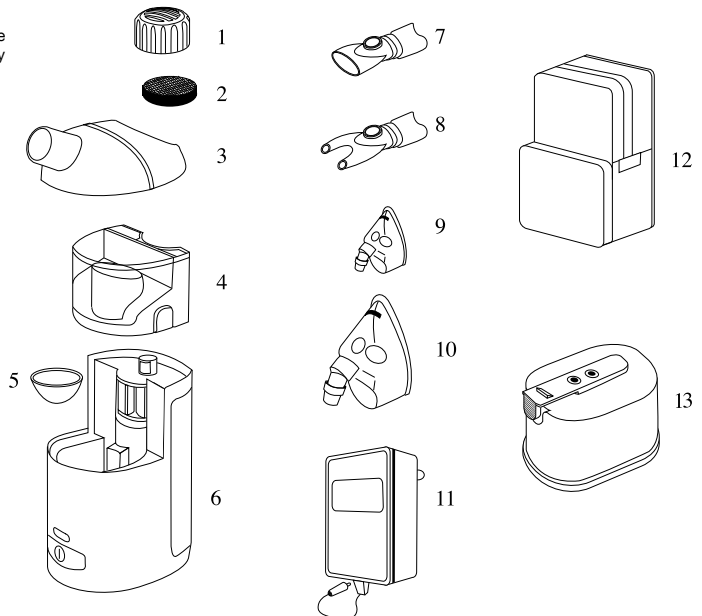
## PRECAUTIONS FOR USING DRUGS

Follow your doctor's instructions concerning the type of drug to use, the dose, and the frequency and duration of nebulising sessions. The Necchi aerosol therapy device has been designed to be used with drugs in an aqueous solution. Drugs that are too viscous or oily will not be nebulised correctly. If a drug cannot be nebulised correctly, try diluting with an equal quantity of physiological solution for aerosol therapy (add 2 ml of physiological solution to 2 ml of an oily drug).

Do not use the device to nebulise substances other than those prescribed by your doctor. Define an alternative therapy with your doctor which you may use in the event of the device malfunctioning or if you cannot use the device (e.g. no electrical power).

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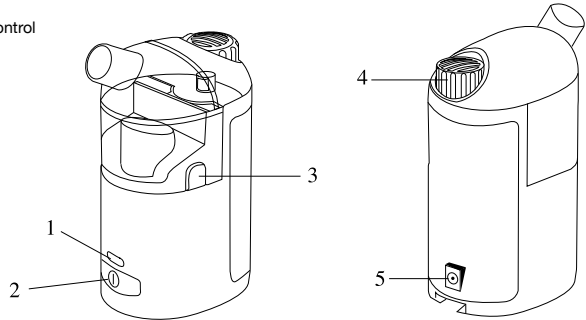
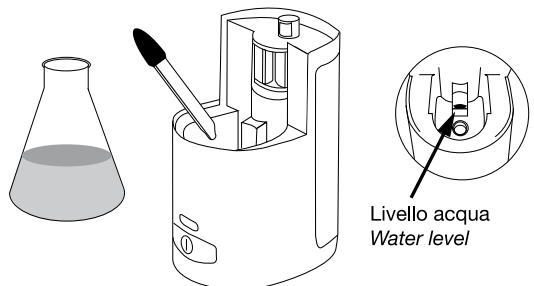
## PREPARING THE DEVICE FOR USE

Before using the device, ensure that all parts have been cleaned correctly as described under "Cleaning".

1. Remove the top cover.
2. Remove the nebulising chamber from the device by pressing the buttons at the sides of the chamber itself.
3. Remove the drug container ampoule from the device.
4. Fill the reservoir with distilled water (available in stores and supermarkets) up to the red level marking (approximately 4 ml). This volume of water is used to transmit the ultrasound waves to the drug and will not be nebulised.
5. Fit the drug container ampoule in its seat, taking care not to damage the ampoule.
6. Pour the drug prescribed by your doctor into the drug container ampoule. If the dosage to be nebulised exceeds the maximum capacity of the ampoule, split and administer the dose over separate nebulising sessions.
7. Refit the nebulising chamber and the top cover on the device. Ensure that the device is sealed hermetically by pressing gently.
8. Fit the mouthpiece, forked nosepiece or one of the masks supplied onto the front of the device.

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## POWER SUPPLY

After preparing for use, connect the device to an electric power supply. The Necchi aerosol therapy device may be used with three different power sources:

### AC/DC TRANSFORMER

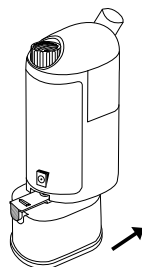
The transformer is designed for household use or for wherever standard mains power is available (different connectors are available for use with different voltages). 1. Plug the transformer into the mains socket, ensuring that the mains voltage matches the specifications indicated on the transformer itself. 2. Connect the AC/DC transformer plug into the power socket on the device.

### RECHARGEABLE BATTERY


The device may also be used with a rechargeable battery. The rechargeable battery provides approximately 45 minutes of autonomy.

- Install the battery, connecting the male fitting on the battery with the female fitting at the base of the device. Slide the battery until it clicks audibly into place. The battery is now installed.
- Remove the battery from the device by releasing the catch.
- Recharge the battery by connecting the AC/DC transformer plug. Minimum recharge time is 90 minutes.


**Note: when charging a new rechargeable battery for the first time, the charge time must not be less than 160 minutes.**

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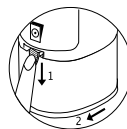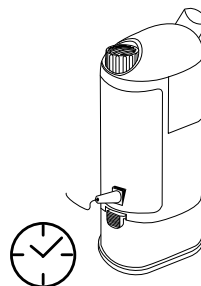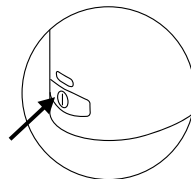
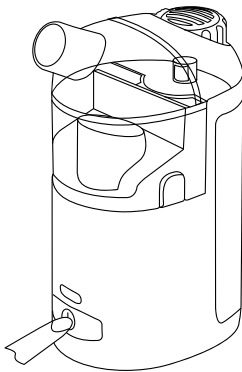
## NEBULISING THE DRUG

1. Sit comfortably in front of the device: switch the device on by pressing the ON/OFF button. A blue light illuminates indicating that the device is functioning normally.
2. The device must be kept upright during nebulisation to ensure that the device itself functions correctly and for maximum treatment efficacy.
3. Inhale the nebulised drug prescribed by your doctor. To facilitate delivery of the drug directly into the bronchi, adjust the aerosol flow speed by turning the flow adjuster knob to the lowest setting.
4. You may notice interruptions in vapour production as the drug in the ampoule begins to run out. You may still continue until no more vapour is produced.

**Important:** as with the majority of aerosol therapy devices, a small amount of drug (approx. 0.4 ml) remains in the ampoule at the end of the treatment session. This is perfectly normal. This remaining quantity of drug, also known as the residual volume, cannot be nebulised.

**N.B:** if slower nebulisation is necessary, press the ON/OFF button again. A green light illuminates to indicate that the device is operating in reduced speed mode.

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## CLEANING

To keep the device in proper working order, it must be thoroughly cleaned on a regular basis.

After each session:

1. Disconnect the transformer plug from the power socket on the device.
2. Remove the nebulising chamber and the top cover from the device.
3. Remove the drug container ampoule and rinse under running water.
4. Empty the transmission liquid reservoir completely.
5. Remove the filter from the cover.
6. Dry the main unit with a soft, clean, moistened cloth.

### **Never clean with benzene or flammable solvents.**

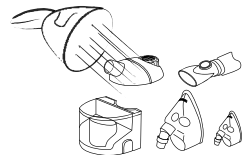
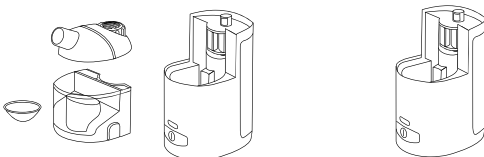
7. Wash all parts of the device, especially those in contact with the drug (top cover, nebulising chamber, mouthpiece, nosepiece and masks), with running water.
8. These parts (with the exclusion of the main unit) may also be rinsed thoroughly under warm water (max. 60°C for 10 min.) and then dried with a clean, dry cloth. Leave to air dry.

## Periodical cleaning

Repeat all the steps given above. You may use a commercially available disinfectant, following the manufacturer's instructions.

The mouthpiece, forked nosepiece, top cover and nebulising chamber may be sterilised by boiling (10 minutes) or by placing in a sterilizer (max 110 °C for 5 minutes). Do not boil or sterilise the main unit or any parts not described above.

For optimum cleaning and to prevent the build-up of residue from compromising the performance of the device, use a solution of water and vinegar (two parts water and one part vinegar). Pour the water and vinegar solution into the reservoir up to the red level marking, fit the top cover and the nebulising chamber (without the drug container ampoule) and run the device for approximately one minute.

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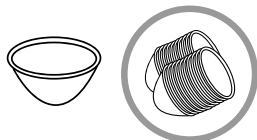
Lack or shortage of water in the reservoir.

## MAINTENANCE

### Replacing the drug container ampoule

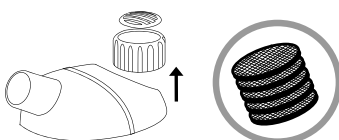
The drug container ampoule must be replaced after every two nebulising sessions. The pack includes 20 replacement ampoules.

### Replacing the air filter

An air filter is fitted to the base of the cover to ensure that the vapour produced by the device does not contain contaminants. How frequently the filter should be changed depends on the conditions in which the device is used (e.g. more or less dusty conditions). However, we recommend replacing the filter after approximately 20 hours of use or when it turns a greyish colour. The pack includes 5 replacement air filters.

#### To replace the filter:

- Remove the top cover from the device. Lift the filter mounting off the top cover.
- Remove the air filter from the mounting and fit a new filter.

5 parti di ricambio  
5 replacement items

## SAFETY PRECAUTIONS

### Overheat protection

The power indicator light is BLUE during normal use. However, in the two conditions described below, the device will indicate a malfunction and will switch off automatically. Should this occur, leave the device off for approximately 45 minutes before switching on again.

Do not obstruct the air intakes and do not place the device on soft surfaces.

### Overheating due to prolonged use

The electronic components of the device are designed to ensure normal operation for a limited period of time (approximately 40 minutes of continuous use). Using for longer periods will damage the protection mechanism and cause the device to switch off.

### No liquid in reservoir

The device switches off automatically if the liquid (demineralised water) in the reservoir is finished or insufficient.

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## ATTENTION - Instructions for a correct disposal of the product

This symbol indicates that the product must not be disposed together with domestic waste after its usage, but in accordance with the legal provisions in force, in order to avoid negative health and environmental implications.

## SPECIFICATIONS

Power supply: 100~240V, 50~60Hz; 12Vdc, 1.2 max.  
Power consumption: max. 16W  
Noise levels: Less than 40dBa  
Frequency: 2.5 Mhz  
Density: 0.5~4.0  
Ampoule capacity: max. 3 ml  
Medium speed: max. 3 ml  
Medium speed: max. 1.0 ml/min (NaCl 0.9%)  
Intermittent use: Hi-16min±3min, Lo-32min±3min  
Dimensions: Approx. 69 mm x 52 mm x 129 mm  
Weight: approx. 219 g (excluding batteries)

## Ambient conditions:

Operating temperature: 10°C~40°C (50°F ~ 104°F); RH 10%-95%  
Storage temperature: -15°C~60°C (5°F~140°F); RH 10%-95%

N.B.: Performance figures given in compliance with the EN 13544-1 regulation may not be correct in the event of use with drugs in suspension or with high viscosities.

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CE 0494 BREMED Ltd. Hong Kong